

## Perching Stool with Loop Back/Arms & Vinyl Padded Seat - 90245

### Assembly Instructions and User Guide

Thank you for choosing this Perching Stool with Loop Arms and Vinyl Padded Seat & Back. This user guide should be read carefully before use as it contains important information about the products safe operation and maintenance.



Product Dimensions	
Width (mm)	560mm
Depth	415mm
Seat Width	330mm
Seat Depth	285mm
Seat height Min	825mm
Seat height Max	965mm
Width Between Arms	510mm
Weight Capacity	165kg (26st)

### WARNING: Max User Weight: 165kg (26st)

### Use of the Perching Stool

- When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the perching stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

### Cleaning

Prior to cleaning the chair, please remove all pin clips and the four extension legs with the ferrules fitted. Clean

these items separately to the main frame.

Clean the upholstery using a non-abrasive soft cloth with a mild detergent. Abrasive cleaners and abrasive cleaning pads will seriously damage the upholstery beyond repair.

The products in this group can be treated with formaldehyde (upholstery only), but it is important that this treatment be carried out by an approved contractor or an experienced person and under suitable conditions.

## **Operation Instructions**

To adjust the height, remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg.

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and that it fits neatly and snugly around the outside leg.

Note the perching stool is designed to slant to the front. Do not adjust any of the legs to make the seat level as this will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool, including fracture of the tubular frame and other sections.

## **Check**

Periodically check to ensure all components are present, correctly assembled and suitably tightened.

## **Disposal**

This product should be able to be recycled at a local recycling centre. The frame is made from powder coated Mild Steel tubes & the backrest and seat are made from PVC vinyl, fire retardant foam & wood. Please ask at your local recycling centre about recycling the pads. Always dispose of products responsibly.

## **Warranty**

This product is supplied with a twelve month warranty which covers any material defects or problems with the manufacture of the product. The warranty does not include wear & tear or any misuse of the product. If you have any problems with this product, please contact your local Ross Care depot or your supplier of the product.