



**Adjustable Headrest Assembly
for Manual Wheelchairs
Part No. 90663**

IMPORTANT










Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.


SYMBOL APPEARING ON THE LABEL OR IN THE INSTRUCTION MANUAL

	Manufacturer's address		Invitation to consult the user manual
	Date of production		Follow the safety instructions
	Batch number		CE conformity
	Product reference		Does not tolerate dampness
	Maximum weight supported		

PRODUCT DESTINATION

The Adjustable Headrest is a medical device. It is a technical aid for the elderly, people with reduced mobility, disabled, injured or who can not easily support their head and/or neck.

PRODUCT DESCRIPTION

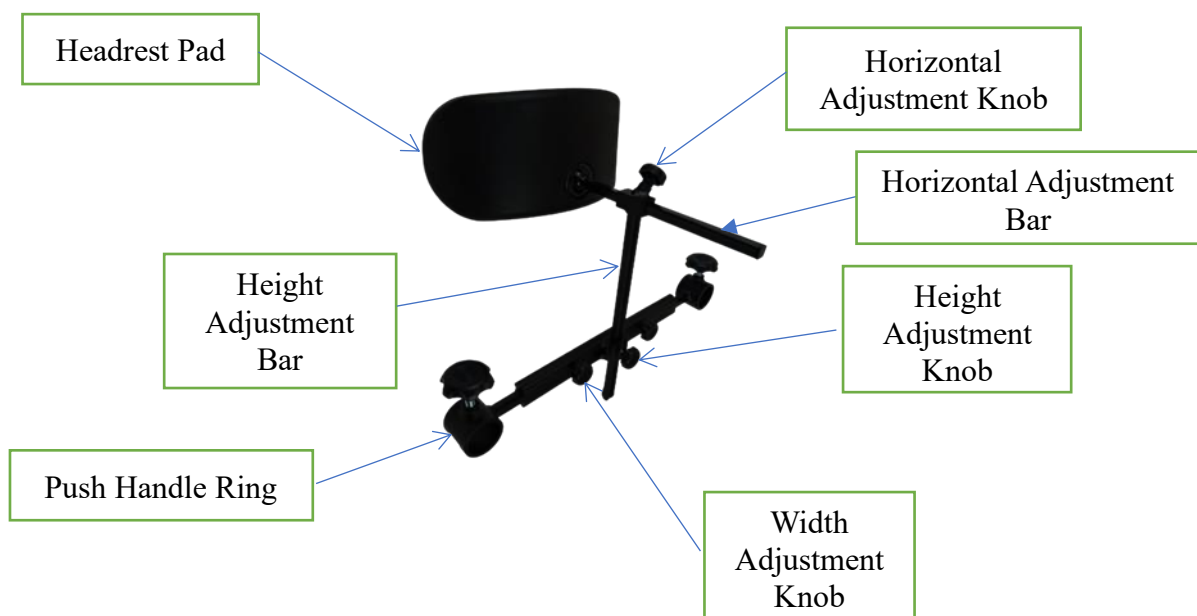

SAFETY PRECAUTIONS



Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 25 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Check the stability of the headrest before use.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING

1. Remove contents from carton
2. Adjust the width of the main bar with Push Handle rings to the correct width to suit the wheelchair that the headrest is being fitted to. **Ensure both sides of the main bar are of an equal width.** (See Image 1 below)
3. Lock the push handle rings to the Push Handles by tightening the knobs above the rings.
4. Lock the width adjustment in place by rotating the width adjustment knobs on each side until they are tight.
5. Screw the Headrest pad onto the Horizontal Adjustment Bar using the screws provided. (See Images 2 & 3 below)
6. Insert the headrest pad & swivel bar into the height adjustment bar tube, with the headrest pad as close to the tube as possible. Tighten the horizontal adjustment bar to a finger-tight position with the Horizontal Adjustment Knob.
7. Place the vertical bar into the height adjustment tube & drop to the required height for the user of the wheelchair. Tighten with the Height Adjustment Knob.
8. Loosen the Horizontal adjustment bar & move the Headrest pad forward until it reaches the most comfortable position for the wheelchair user. (See Image 4 below) Tighten the Horizontal Adjustment Knob to lock the pad in position.
9. If required, tilt the pad to a position that is comfortable for the user.



Image 1



Image 2



Image 3



Image 4

CARE AND MAINTENANCE



Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The product can be cleaned in an aqueous decontaminant machine at 80°C.
- It is recommended that the device is serviced at least annually.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE



Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything.)

PERIOD OF USE

The life expectancy of this product is 6 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL


Overall Dimensions	Width 48 cm x Depth 33.5 cm x Height 38 cm
Headrest Pad Material	Polyurethane & Wood
Weight of Assembly	1.4 kg
Maximum supported weight	25 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. Integrity will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.



Integrity Mobility Solutions
74 St. Georges Road
Bolton BL1 2DD

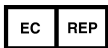

Ives MD EU Rep. Ltd
44 Rosses Stables,
Ballymena, Co. Antrim
BT42 2SF

