




harvesthealthcare®

equipped to care

INSTALLING THE 200MM BED EXTENSION

If the bed is already assembled.


Remove the side rails and disconnect the lift system from the foot end of the bed as described in the IFU supplied with the bed.

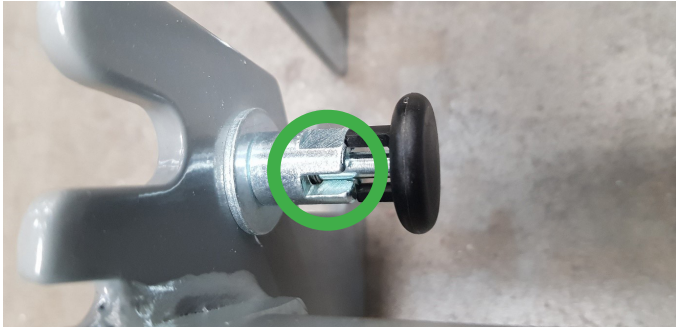

1. Disengage both locking pins on the foot end of the sleeping platform.



2. Attach the extension to the foot end of the sleeping platform.

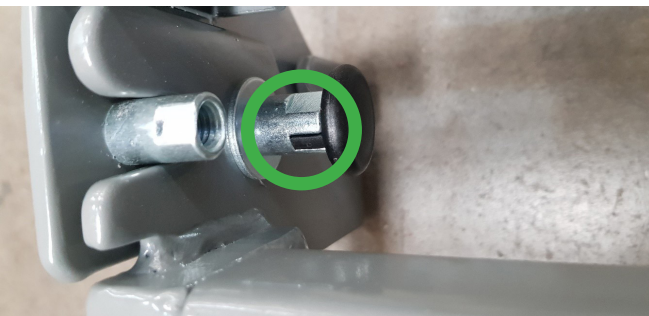

3. Engage the both locking pins on the platform.



Sheaf House, Bradmarsh Business Park, Bradmarsh Way, Rotherham S60 1BW
T +44 (0)1709 377172 F +44 (0)1709 377173 E sales@harvesthealthcare.co.uk

INSTALLING THE 200MM BED EXTENSION



4. Attach the lift system to the sleeping platform/ extension and re-engage the locking pins on the lift system.

5. If side rails are required re install the extended side rails as described in the IFU supplied with the bed.

Please Note:

- Harvest Healthcare recommend this procedure is carried out by competent persons.
- Please carry out a risk assessment to ensure the equipment is suitable for the user.
- Do not fit the extension with a person in the bed.
- Check all fixings for security at regular intervals.
- If you have and doubts about any part of the procedure contact harvest healthcare Ltd.
-

Product Codes:

- HLDW111: Extended Side Rails
- HLDW123: Wooden Side Rail Length Extension With Platform
- HLDWC123: Community Bed Length Extension With Platform
- HLDU123: Aluminium Side Rail Length Extension With Platform
- HLDU123.12: Ultimate 1200 Bed Length Extension With Platform
- HLDU123.14:Ultimate 1400 Bed Length Extension With Platform

REVISION 1 MC-14/05/2019