

## How to make up TECcare CONTROL disinfectant solution

Use for general cleaning / disinfection on all hard and soft surfaces as well as for decontamination of medical equipment after each use. For correct dosing please refer to chart below:-

Amount of cold, clean water	Amount of TECcare CONTROL Concentrate required	...or if using e12ml Pelican pump (1 x e12ml dose = 1 full pump depression)
1 Litre	12ml	1 dose
2 Litres	24ml	2 dose
3 Litres	36ml	3 doses
5 Litres	60ml	5 doses
10 Litres	120ml	10 doses

### ALWAYS...

- Wear appropriate protective clothing (PPE) gloves, apron etc., and refer to company's Infection Prevention/Control Policy.
- Read precaution information on the product label before use.
- Check expiry date on product label before use.
- Change solution when it becomes dirty.
- Use clean water to dilute product.
- Use with disposable cloths or mops to wipe over surfaces or floors.
- Discard any remaining solution.

### NEVER...

- Mix with any other cleaning agent or any chemical.

### PRECAUTIONS...



Keep away from children.



If in contact with eyes rinse with clean water.



After use rinse hands with water and dry thoroughly.



Avoid prolonged use if you have sensitive or damaged skin.



Do not swallow. If consumed seek medical advice.

**Read Safety Data Sheet before use** (available on request)



- If any irritation persists consult doctor.
- Keep only in the original container.

If in doubt about when to use this product contact your Supervisor or your Infection Prevention/Control Team.